




Important! This quick reference guide does not replace the user manual which can be downloaded at www.humancaregroup.com. Read all instructions before use!


Components

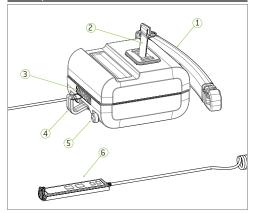

- 1 Telescopic Hanger Bar
- 2 Lift Belt
- 3 Secondary Panel

Important! This overhead lift has been designed and tested for indoor use with one patient at a time. This product must not be immersed in water. Read all instructions for full product information!

Hand Control Functions

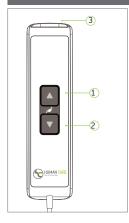

Charging the Lift

Flashing Red - Charge lift immediately!

1 Up Button 2 Down Button 3 USB and Charge Port

Important!

The covers for the USB input and the charging input (3) must be closed for the hand control's water protection (IPX5) to be valid.

1 Connect charger plug to

2 Connect plug to charger and

insert plug into wall to charge.

connected to an earthed socket.

Note: If the emergency

stop is activated, the lift will

The charger must only be

handset.

not charge!

Important! The lift can only be used with a Human

Care charger and must be disconnected to use the lift.

Using the lift



Emergency Stop Activation Press down on red button to activate

Emergency De-Activation Turn red button clockwise.

Important! The emergency lowering button must only be used in emergency situations.




Suspending the lift

Place the lift on a suitable surface and feed out belt by gently pulling out belt while at the same time pressing the "Down" button on either the hand control or secondary panel.

Hook lift belt onto distance strap so that it is held firmly in position (easiest to hook from above at an angle as shown). Ensure the lift belt is firmly in place before use.


Telescoping hanger bar

Three adjustable positions (40, 50 and 60cm) - turn hook counter-clockwise while gently pulling out. Hook will fall into each position. Repeat until desired distance is achieved.

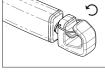

Slings

Attach head sling loops to carry bar first, then leg straps to leg hooks. Ensure the straps are secured beneath the safety pin.

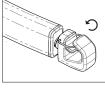



Proper lifting techniques

Do not use the telescoping bar to move the patient and the lift. Do not hold onto the hooks while performing a lift. Once the patient is comfortably secured in sling, raise lift, hold sling and position/move patient.

General Care



Charge lift every night.



Wipe down with slightly dampened drying cloth.



Disinfect with DuPont™ Virkon®.



Never use strong detergents on lift.





Activate emergency stop when not in use.



Important! Do not use strap to transport!

Lift must always be stored or moved in horizontal position. Rest on cart to move.

Inspection

Annual Inspection - The lift must be inspected at least once per year by a gualified technician certified by Human Care. Monthly Inspection - Check lift belts for damage or wear. Feed entire length to ensure proper functioning. Daily Inspection - Always inspect the lift for visible signs of damage before use. Ensure the lift is correctly mounted on the distance strap or rail roller. Always check slings and belts

for signs of wear before use. Do not use if wear evident.

Troubleshooting


If the indicator light is not on, check the following:

- Mains switch is on
- Batteries do not require charging
- Emergency stop button is not pushed in (deactivate by turning clockwise)




If the indicator light is flashing, check the following:

- Check the hand control button functions up / down when depressed (replace if necessary)
- When pressing up, the lift is not already in its maximum upper position
- When pressing down, the lift is not already in its maximum lower position

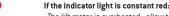


- The lift motor is overheated - allow to cool

- The lift is overloaded - lower using emergency lowering button

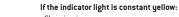



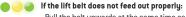


- Pull the belt upwards at the same time as you depress the Down button on the hand control / lift panel



If the indicator changes to flashing red:

- The batteries need charging, patient may be lifted a few more times

Secondary Panel Functions

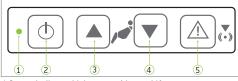

- 1 Status Indicator Light
- 2 Power On/Off
- 4 Lower Lift 5 Emergency Lowering

4 Leg Hook

5 Emergency Stop

6 Hand Control

3 Raise Lift

Status Indicator Lights



Flashing Yellow On/Ready for Use Charge Battery Warning

- Flashing Red On/Ready for Use Low Battery
- Constant Green Lift in Use Lift Loaded

QUICK REFERENCE GUIDE





Service Warning

Flashing Red/Yellow Service Passed Battery Low

Safety Catch Engaged Troubleshooting!



Flashing Green/Yellow Perform Inspection



Models: 55030, 55040, 55050

Human Care HC AB. All Rights Reserved.

(2)





